

# Comfort Marker 2.0

Easy application of patient reference points

*Medical*

---

*Precision b.v.*



**CIVCO**  
Radiotherapy™  
a better way

# Easy Application of Patient Reference Points

Medical

Precision b.v.

## Comfort Marker 2.0 A New Level of Comfort

Improve your patient's radiotherapy journey and gain clinical efficiencies with the Comfort Marker 2.0, developed by Medical Precision B.V. This revolutionary reference point device brings a new level of comfort to the start of the patient's radiotherapy treatment.

The easy-to-use system is designed specifically for use in radiotherapy using technology popular within the permanent make-up industry. The device is carefully engineered to apply reference marks with controlled depth placement, resulting in less pain to the patient compared to other marking methods. Points may fade over time or are easily removable with a simple laser session after the patient's treatment is complete.

### Patient Benefits

- Gentle and controlled placement of pigment into skin with less pain to the patient
- Places pigment at depth of 0.2–0.7 mm compared to common practice of 2–3 mm
- Reference points may fade over time or are easily removable
- Uses high-quality ink

### Clinician Benefits

- Less pain keeps the patient relaxed and more cooperative throughout the process
- Fast, sterile and disposable safety needle module minimizes risk of needle stick incidents
- Safe and simple operation with clear indication of depth settings
- Charged device allows portability in the CT room

### Ordering Information

Comfort Marker 2.0 System, Qty 1	MPCM2010INT
Patient Marking Set. Includes: Safety Needle (Qty 50), Ink Cup (Qty 50), Black RadSafe Pigment (Qty 50)	MPCM4022INT



Safety Needle, Ink Cup, Pigment

CE MARKED.  
THIS BROCHURE IS INTENDED FOR MARKETS OUTSIDE THE U.S.



## Simple Workflow



Watch the Comfort Marker 2.0 Instructional Video on CIVCO RT's YouTube channel!

1

Place Ink Cup onto the Device.  
Squeeze Pigment into the Cup.



2

Place the Safety Needle into the Pen.



3

Turn on the Control Unit.




4

Choose the depth and absorb the Pigment (safety needle will light up the same color as depth chosen).



5

Place the reference point precisely.

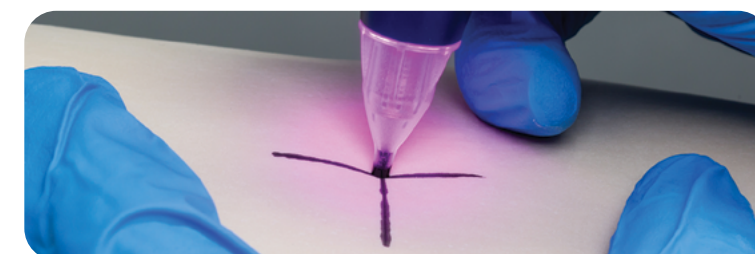

PLEASE SEE INSTRUCTIONS FOR USE FOR ADDITIONAL INFORMATION.

### What European Radiation Therapists in the Netherlands are saying:

"The cosmetic impact of the Comfort Marker's points are much less than the traditional points of 2 to 4 mm diameter placed manually. Points now are small and not dominantly present. For radiotherapy however they are very well visible reference points. What we like also is that everything is certified at Medical Grade, and this is something our hospital values as well..."

"...Placing the points with the Comfort Marker is much more comfortable for the patient... This makes our work much easier: the patient simply remains much more calm. Finally, we as professionals are happy with a system that really prevents sticking incidents. It is safe."



# CIVCO

Radiotherapy™



**Global Sales Office**  
1401 8<sup>th</sup> Street SE  
Orange City, IA 51041, USA  
800.842.8688 | +1 712.737.8688  
[info@CivcoRT.com](mailto:info@CivcoRT.com) | [www.CivcoRT.com](http://www.CivcoRT.com)